ME Medical™ Professional Clipper Model SC004**SAVE THESE INSTRUCTIONS****Important Safety Instructions**

When using this clipper, basic precautions should always be followed. Read and understand all safety information before using.

INTENDED USE

- The ME Medical Professional Clipper, SC004, is a cordless, rechargeable, battery-operated clipper which is comprised of a handle, a charge stand, and single-use blade assembly.
- The ME Medical Professional Clipper, SC004, is intended to remove body hair and head hair from patients in preparation for any procedure requiring hair removal.
- By replacing the common razor, this product will minimize the trauma associated with conventional shaving techniques.
- With proper use, the ME Medical Professional Clipper, SC004, removes hair quickly and thoroughly; it has sufficient power to remove coarse hair, yet is gentle enough for fine hair. It works well on all types of hair, wet or dry, even in delicate areas.
- The clipper has been designed and tested for use only with the Applied Parts ME Medical Professional Clipper disposable blade assembly, SC002 Universal blade, SC003 Neuro blade and SC008 Precision blade.

WARNING

To reduce the risks associated with fire and explosion, which if not avoided could result in serious injury or death and/or property damage:

- Do not operate clipper where aerosol (spray) products, flammable anesthetic mixture with air, nitrous oxide or oxygen administering equipment other than nasal or mask types are being used.
- Emergency Medical Technicians (EMTs), do not use clipper on a patient whose clothing is saturated with flammable solutions such as gasoline, fuel oil, solvents, etc.

- Avoid using around flammable alcohol or similar cleaning solution. If alcohol is used to clean the clipper, assure that the clipper and surrounding area is ventilated of fumes before plugging in the drop-in charger or switching on the clipper.

To reduce the risks associated with hazardous voltage, which if not avoided could result in serious injury or death:

- For recharging the battery, only use the detachable supply unit provided with this appliance.
- Do not attempt to plug or unplug the drop-in charger from an electrical outlet with wet hands. Unplug the drop-in charger before cleaning.
- Do not place or store a drop-in charger where it can fall or be pulled into a tub or sink of water.
- Do not retrieve a drop-in charger that has fallen into or is submerged in water or other solutions. Unplug it from the wall outlet immediately.
- Never charge the clipper if the drop-in charger has a damaged cord or plug; if it is not working if it has been dropped or damaged, or if the drop-in charger has been dropped into water.
- Do not put in fire or damage the battery pack during disposal as it may burst or release toxic materials.
- Do not short circuit the battery pack by bridging the battery contacts with a metal or conductive part as it may cause burns.
- Do not attempt to replace the battery pack. This could result in the risk of fire or electric shock.

To reduce the risks associated with cross contamination, which if not avoided could result in serious injury or death:

- Do not reuse the blade assembly. Single patients use only.
- Always clean the clipper handle between patients.

CAUTION

To reduce the risks associated with fire, and hazardous voltage, which if not avoided may result in minor or moderate injury and/or property damage:

- Do not sterilize the clipper body, drop-in charger, or blade assembly.

- Only use attachments recommended by the manufacturer.
- Always store the clipper and drop-in charger in a moisture-free area.
- Do not attempt to replace the battery in the clipper.
- Do not tamper with the device's internal components. There are no user serviceable parts inside this device.
- Do not charge the clipper outdoors.
- Keep the charger cord away from heated surfaces.
- See rating plate on drop-in charger unit for proper wall outlet voltage.
- This device may cause (EMI) Electromagnetic Interference. If this occurs, separate product from affected device.

To reduce the risk associated with environmental contamination, which if not avoided may result in minor or moderate injury:

- At the end of product life, dispose of all components in accordance with your governmental regulations.
- Dispose of batteries properly in accordance with your local, state, and/or government requirements.
- Do not tamper with the device's internal components. There are no user-serviceable parts inside this device.

To reduce the risks associated with skin nicks and lacerations, which if not avoided may result in minor or moderate injury:

- Do not use the clipper with a damaged or broken blade assembly.
- Do not clip in a raking manner as this technique may damage the patient's skin.
- Blade temperature can reach upwards of 50°C after prolonged use.

Explanation of Symbols

Attention: See Instructions For Use

Do Not Reuse

I Equipment Switch 'On'

O Equipment Switch 'Off'

The disposal of Li-ion batteries as municipal solid waste is prohibited.

Recycle in accordance with your local, state, and/or government requirements.

Applied Parts according to IEC 60417-5333: blade assembly SC002
Universal blade and SC003 Neuro blade.

Transport conditions: -20°C to +35°C; 10% to 100% RH, 50 to 106 kPa

Operation conditions: +0°C to +40°C; 30% to 75% RH, 70 to 106kPa

INSTRUCTIONS FOR USE

To Attach a Disposable Blade to the Clipper Body:

Make sure that the ON/OFF switch is in the OFF position. After removing the lid of the blade assembly packaging and without touching the blade, hold the clipper body over the blade assembly. Slide the new clipper blade assembly into position until it clicks.

To Use the Clipper:

The patient's skin should be clean. To turn the clipper on, simply push the ON/OFF switch above the LED battery gauge on the clipper to the ON position. The power indicator will light up.

Use short, gentle strokes, holding the clipper blade assembly at a flat to slight angle to the patient's skin. For best results, clip against the direction of the hair growth. For contoured areas, stretch the skin taut while clipping. The Clipper head can be twisted 45° to the left and to the right and be locked in to reach difficult to reach areas of the body. There are several ways to grasp the clipper. Determine the most comfortable technique that allows you to keep the clipper blade assembly at a flat to a slight angle to the skin. This technique can be performed in a push or pull motion. Never rake the clipper over the skin.

To Change Disposable Blade:

Push the ON/OFF switch into the OFF position. Hold the clipper body in one hand. Position the clipper over an appropriate waste receptacle with the blade facing downward. With your thumb, push the blade off the clipper body into waste receptacle.

NOTE: To reduce cross contamination between patients, remove the clipper blade and discard appropriately after each use.

Care and Cleaning of the Clipper Body:

The housing of the clipper body is made of durable plastic and is water resistant. It may be wiped off with a damp cloth containing a small amount of alcohol, disinfectant, or a mild detergent solution; It can also be rinsed under running water or soaked in a high-level disinfectant solution. Always follow the manufacturer's instructions for high level disinfection (per ANSI AAMI ST58:2005 Chemical sterilization and high-level disinfection in health care facilities). Allow to dry completely before placing clipper back into the charger or reusing.

Do not sterilize the clipper body or blade.

Charging the SC004 Battery-Operated Clipper:

To charge the clipper:

1. Push the ON/OFF switch to the OFF position.
2. Clean the clipper body before returning to the charger, making sure the clipper and charger contacts are clean and free of hair or other contaminants.
3. Connect the cord of the charger to an appropriate electrical outlet, making sure the cord is not pinched or kinked.
4. To be certain that the SC004 clipper is fully charged, plug it in for 3 hours before its initial use.

WARNINGS

- Do not put in fire or damage the battery pack during disposal as it may burst or release toxic materials.
- Do not short circuit the battery pack by bridging the battery contacts with a metal or conductive part as it may cause burns.
- Do not attempt to replace the battery pack. This could result in the risk of fire or electric shock.

CAUTION

- Dispose of batteries properly in accordance with your local, state, and/or government requirements.
- This appliance contains batteries that are non-replaceable.
- Do not tamper with the device's internal components. There are no user-serviceable parts inside this device.

Components and Accessories

Components:

REF SC004	Professional Clipper
REF SC002	Universal Clipper Blades
REF SC003	Clipper Neuro Blades
REF SC008	Clipper Precision Blades

Accessories:

REF SC010	Charging Station for Prof. Clipper (EU plug Type C)
REF SC010-U	Charging Station for Prof. Clipper (USB-A plug)

Configurations:

REF SC014	Prof. Clipper REF SC004 incl. Charger Stand REF SC010
-----------	---

Technical Specifications

Motor Voltage: 3.7 volt DC

Current without load: 0,18A

Current with full load: 0,7A

RPM: 6'500

Power Adapters Charge Circuits: All power adapters/charge circuits are IN PUT:100-240~ 50/60Hz 0.4A OUT PUT:5.0Vdc 2A

Charging Time: 2 hours

Battery Operating Time: 150 minutes

Battery: Li-ion 3.7V ICR 18650MAB 1800mAh

Clipper Weight (with blade): 142 grams (without blade) 132g

Clipper Dimensions (with blade): 158x46,5x40,5mm

Guidance and Manufacturer's Declaration

Below cables information are provided for EMC reference.

eg.

Cable	Max. cable length, Shielded/unshielded		Number	Cable classification
AC Power Line	1.5m	shielded	1 Set	AC Power

Important information regarding Electro Magnetic Compatibility (EMC)

This electrical medical equipment needs special precautions regarding EMC and put into service according to the EMC information provided in the user manual; The equipment conforms to this IEC 60601-1-2:2014 standard for both immunity and emissions. Nevertheless, special precautions need to be observed:

- The equipment with no ESSENTIAL PERFORMANCE/ is intended used in Home healthcare environment
- **WARNING:** Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally”.
- The use of accessories, transducers, and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- **WARNING:** Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the SC001, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.”
- **STATEMENT:** For its operation, the equipment has wireless communication function, it includes RF transmitter and receiver, 2.4GHz, Pulse modulation.

EMI Compliance Table (Table 1)

Table 1 - Emission

Phenomenon	Compliance	Electromagnetic environment
RF emissions	CISPR 11 Group 1, Class A	Professional healthcare facility environment
Harmonic distortion	IEC 61000-3-2 Compliance	

Voltage fluctuations and flicker	IEC 61000-3-3 Compliance	
----------------------------------	-----------------------------	--

EMS Compliance Table (Table 2-5)

Table 2 - Enclosure Port

Phenomenon	Basic EMC standard	Immunity test levels
		Professional healthcare facility environment
Electrostatic Discharge	IEC 61000-4-2	±4 kV contact ±8kV air
Radiated RF EM field	IEC 61000-4-3	3V/m 80MHz-1GHz 80% AM at 1kHz
Proximity fields from RF wireless communications equipment	IEC 61000-4-3	Refer to table 3
Rated power frequency magnetic fields	IEC 61000-4-8	30A/m 50Hz or 60Hz

Table 3 – Proximity fields from RF wireless communications equipment

Test frequency (MHz)	Band (MHz)	Immunity test levels
		Professional healthcare facility environment
385	380-390	Pulse modulation 18Hz, 27V/m
450	430-470	Pulse modulation 18Hz, 28V/m
710	704-787	Pulse modulation 217Hz, 9V/m
745		
780		
810	800-960	Pulse modulation 18Hz, 28V/m
870		
930		
1720	1700-1990	Pulse modulation 217Hz, 28V/m
1845		
1970		
2450	2400-2570	Pulse modulation 217Hz, 28V/m
5240	5100-5800	Pulse modulation 217Hz, 9V/m
5500		
5785		

Table 4 – Input a.c. power Port

Phenomenon	Basic EMC standard	Immunity test levels
		Professional healthcare facility environment
Electrical fast transients/burst	IEC 61000-4-4	±1 kV 50kHz repetition frequency
Surges Line-to-line	IEC 61000-4-5	±1 kV, ±2 kV
Conducted disturbances induced by RF fields	IEC 61000-4-6	3V, 0.15MHz-80MHz 80%AM at 1kHz
Voltage dips	IEC 61000-4-11	0% UT; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°
		0% U _T ; 1 cycle and 70% U _T ; 25 cycles Single phase: at 0°
Voltage interruptions	IEC 61000-4-11	0% U _T ; 250/300 cycles

Table 5 – Signal input/output parts Port

Phenomenon	Basic EMC standard	Immunity test levels
		Professional healthcare facility environment
Conducted disturbances induced by RF fields	IEC 61000-4-6	3V, 150kHz-80MHz 80%AM at 1kHz

Warranties/ME Medical Liability

Limited Warranty

This warranty is in lieu of any other warranty express or implied except as indicated below: This

ME Medical Professional Clipper is warranted against any defects in material or workmanship for a period of 2 years from the date of purchase. If this surgical clipper is determined to be defective within the warranty period, it will be replaced without charge by ME Medical. This warranty does not cover damage caused by misuse, negligence, accident, abuse or alteration, modification of the units by anyone other than ME Medical or affixation of any accessory not manufactured by ME Medical, or use of the surgical clipper with current or voltage other than that specified in this instruction insert.

Replacements after the warranty period will be made and charged to the customer on the basis of rates which are available upon request.

EXCEPT FOR PERSONAL INJURY, ME MEDICAL IS NOT LIABLE IN EITHER TORT OR CONTRACT FOR ANY LOSS OR DAMAGE, DIRECT, CONSEQUENTIAL, OR INCIDENTAL ARISING OUT OF THE USE, MISUSE, OR INABILITY TO USE THIS PRODUCT. All implied warranties are limited in duration to 2 years from date of original purchase.

CE

Made in China for **ME Medical AG**, via Valresa 52C, 6645 Brione s. Minusio, Switzerland

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Made in China for ME Medical AG, via Valresa 52, 6645 Brione s.M., Suiza